



Facial Treatments




THE BEST FACIAL TREATMENT YOU'LL EVER HAVE!

Experience the new OXY TOUCH

7-in-1 Advanced Facial Exfoliate+infuse+Oxygenate+Hydrate

Scientifically proven oxygenation
Reduce appearance of pores
Renewed brightness and glow
Increased collagen
Reduced brown spots
Reduce appearance of wrinkles
Skin hydrating

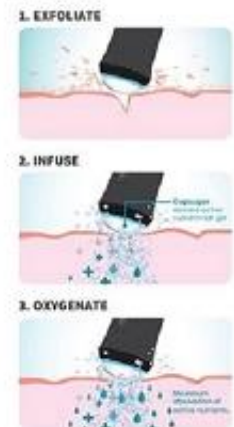

Accessories



OXY-effect handle



4D-Circle RV handle



Sprayer



Cool & Heat work handle



Small Micro vibration work handle



Big Micro vibration work handle



Aqua peeling clean handle



Aqua peeling clean cap

Aqua Peeling

Bottle No. 1: It is a nutrient solution used to soften cuticles and open pores.

Bottle No. 2: It belongs to cleaning liquid and is used to remove blackheads, pimples and whiteheads.

Bottle No. 3: It is a moisturizing liquid, which is used to nourish the skin and replenish water while shrinking pores.

S1: Dotted water distribution.


S2: To remove blackheads in the form of rotation or lifting.

S3: Hydrates the skin.



Operation of Oxycarbon Stone

1. Apply even carbonated gel to the treatment area.
2. Press the product onto the handle.
3. Adjust the intensity of the interface, choose red or blue light, click start on the interface, and click the start button on the handle.
4. The handle is in touch with face, and a large amount of foam appears on the face (10-15min) by rotating in a circle or lifting.
5. A slight circle around the cheekbones and corners of the eyes.

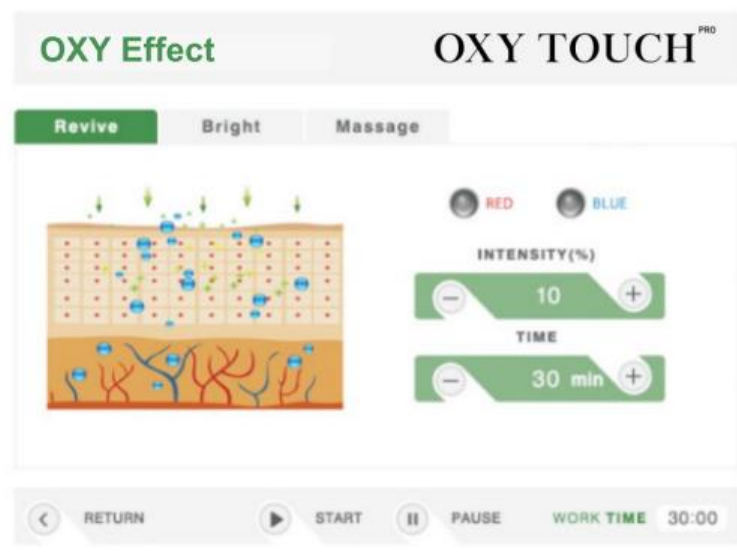


Operation of 4-Polar RF

1. Apply enough medium (gel, product) to the care area.
2. Adjust Energy level.
3. The working head of the handle is in touch with the nursing face in a circle or lifting. (Remember, you can't stay at a certain part without moving)
4. Working time: 15min-20mins is enough.



Operation of Cool&Heat Handle

1. Smearing gel or effect products.
2. Click on the interface "heating" and "temperature": 39 degrees-43 degrees, the actual situation is adjusted according to customer feedback.
3. Slowly circle or lift the handle on the face to open the pores to introduce the product, or wrap the wet tissue on the work head of the handle, first apply hot compress on the care area, and then lift the skin (15min-20min)
4. Click "Pause" and click "Cool" to start, the nursing part will shrink the pores.(5mins-10mins)

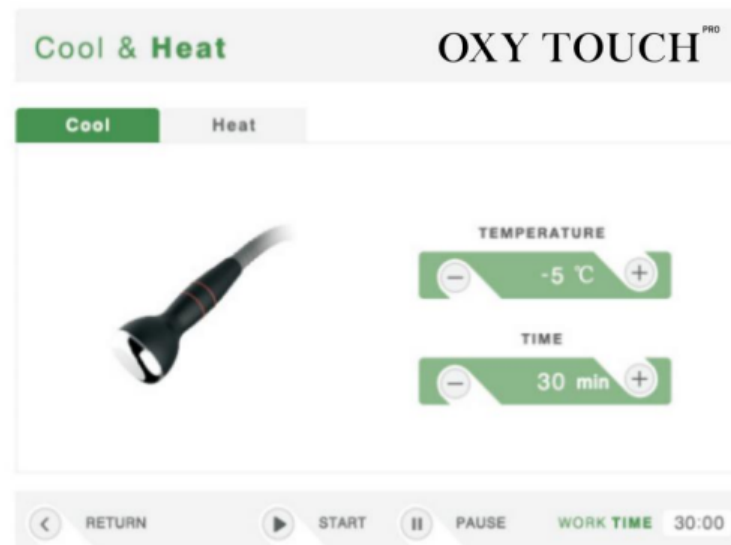

Operation of Ultrasonic Handle

1. Apply media or functional products (water, milky) to the care area
2. The mode, that is, the frequency is adjusted according to the actual situation (the slower the frequency, the stronger the penetration), and the intensity is adjusted according to the actual situation.
3. Click "Start", the handle head is next to the nursing part to do slow moving circle or lifting technique (15min-20min).
4. Eye operation time is 5min-8mins.



Operation of Sprayer

1. Introduce 3/2 of the water-like product into the container on the handle, and turn the screw button at the end of the handle to adjust the amount of oxygen ejected.
2. Click the trigger on the lower side of the handle and spray from top to bottom in the nursing area. So as not to enter the customer's eyes and nose. (15min-20min)

